# C3567

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## **ELECTRIC COUCH INSTRUCTIONS**

Ecopostural thanks you for buying our products. The item you have just purchased has been manufactured according to high quality and security standard. Follow the below-mentioned instructions carefully in order to be able to make use of it for a long time. Click on to the **TECHNICAL INFORMATION** section in our website **[www.ecopostural.com](http://www.ecopostural.com)**

**Ecopostural tables have been conceived to support the patient during an examination and/or treatment session by healthcare personnel or under his supervision in those fields such as physiotherapy, rehabilitation and related disciplines.**

|                                                                                                                                                                                                                   |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Our company will decline any responsibility for a product damage occurred if ignoring our operating instructions. We will not be liable for any accident or harm resulting from an undue or negligent use.</b> |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

## INDEX

### Symbols \*

1. General instructions
2. Operating instructions
  - 2.1 Set Up
  - 2.2 Load capacity
  - 2.3 Height adjustment
  - 2.4 Sections adjustment
  - 2.5 Accessories adjustment
3. Maintenance
  - 3.1 Metal frame
  - 3.2 Elevation system
  - 3.3 Section adjustment mechanisms
  - 3.4 Upholstery
4. Guarantee
5. Technical features
6. FAQ about motors and upholstery

## SYMBOLS

Follow the instructions of this user manual.

Read carefully before the first use and observe the operating instructions and the preventive security measures.

**Warning or caution:** This sign alerts about a possible hazard which must be overcome so as to avoid:

- a. Serious or even potentially deadly injuries for the patient.
- b. Mild or moderate severe injuries for the patient.
- c. Material damages to the equipment.

If the use of this device causes or contributes to the presence of an undesired event, such as death or serious injury to the user, the manufacturer **MUST** be informed immediately via email [vigilanciaps@ecopostural.com](mailto:vigilanciaps@ecopostural.com) or the distributor. In the European Union, the user must also notify the competent authorities of the member state in which he is located.

Avoid sitting down on the table headrest.

This product meets the Medical Device Regulation from the European Parliament and the European Council dated 5th April 2017: **(EU) 2017/745**

States the item is a Medical Device

Manufacturer's name, address and manufacturing date.

Refers to the model number

Unique Device Identifier

Manufacturing date

Lot number

Equipment with double isolation IEC 60417-5172

Expendable electrical components that can be recycled

Stands that the electric and electronic components of the device can be recycled and must be disposed of separately.

Waste separation of used products and packaging allows the materials to be recycled and used again. Re-use of recycled materials helps prevent environmental pollution and reduces the demand for raw materials.

\* according to models

## 1) GENERAL INSTRUCTIONS

- Please read carefully the operating and safety instructions before using our products. These instructions must be adhered to. Keep the instructions booklet for further check-ups.
- Familiarize yourself with the precise use of the item.
- If the couch is handled by unqualified people in an inappropriate way or not according to the intended purpose, its use may be dangerous. Be careful when handling the couch as there is a risk of being trapped by the mobile parts and because of sudden movements.
- Regularly check the couch entirely to find any sign of malfunction or damage. Periodically check mains cable and plug. We recommend to handle them with the utmost care and to avoid the bending of the cables.
- Every single part or component of the table follows a thorough quality control thus ensuring a smooth finish. Nevertheless, proceed with caution when handling the table because there might be any corner or protruding part which might result unintentionally harmful.

**When raising or lowering the stretcher or when operating its modules, MAKE SURE THAT NEITHER THE PATIENT NOR THE THERAPIST ARE TRAPPED** between the moving parts of the lifting mechanism by hands, feet or any other part of the body or clothing. Be careful, since there is a risk of entrapment between mobile and fixed parts of the stretcher, or, and due to unexpected movements when activating its modules. Only operate the stretcher when there is no possibility that any part of the body could be trapped and injured while moving the stretcher up/down or activating the tilt of the component modules.

**Pay special attention in the case of children and/or unauthorized personnel. Do not leave children alone or unattended in the area around the stretcher. Maintenance or cleaning tasks must be carried out only by authorized personnel and with the stretcher drive systems properly blocked.**

- Worn or damaged pieces must be solely replaced by original pieces by the manufacturer depending on the table model.
- All components parts or accessories should be fitted or fixed in according to the special manufacturer's instructions. Any spare / replacement component part different to the manufacturer's could be dangerous for the people involved in the table handling.

**\* Notice:** The use of non original pieces or the alteration of the item will incur in a loss of validity of the manufacturer's guarantee, instructions and Declaration of Conformity.

**Ecopostural reserves the right to modify its products and make changes in technical data. Actual dimensions and weights may vary from the values indicated in this catalogue:**

**dimensions  $\pm 5\text{cm}$  / weight  $\pm 5\text{kg}$  / movements  $\pm 10^\circ$ .**

**Our products are handcrafted and made-to-order upon the customer preferences.**

**Consequently, we won't accept any cancellation, return or refund of any Ecopostural product because there's been a previous choice for a specific size, colour...**

**The design of the different sections and tilting angles obeys to the standards of specific profesional areas such as the physiotherapy, rehabilitation and related disciplines, hence the specific dimensions conforming to these practices.**

**ATTENTION: Be careful, as there is a risk of entrapment between moving and/or fixed parts and due to unexpected movements. Only operate the stretcher when there is no possibility that any part of the body could be trapped and injured while moving the stretcher up/down or activating the tilt of the component modules.**

## 2) OPERATING INSTRUCTIONS

### a) **SET UP**

The table weighs approximately 100 kg so at least two people will be required to unpack and install it. To raise the stretcher, grasp the table frame located under the table upholstery. Avoid raising the table by grasping the upholstery or the lifting mechanism as these could be damaged. Follow instructions at all times to avoid injury and always observe the safety procedures to move or lift the stretcher.

The table must lean on a solid and level ground.

The chair must lean on a solid and level ground.

**Notice:** *in certain floor surfaces the table legs might leave marks after a prolonged contact. Check the technical specifications of the floor manufacturer to avoid any trouble.*

In order to move the table around, you will need to release the **castors** by pressing the locking levers downwards. To lock the castors, simply lift both locking levers upwards with your foot.

These castor wheels are only suitable for moving the table around when required, **NOT intended for patient transportation.** Before the patient lies down **MAKE SURE THE CASTORS ARE LOCKED.**

### b) **LOAD capacity**

This item has been tested to support a **MAXIMUM STATIC WEIGHT of 135 Kg** evenly distributed all over. *\* In case this weight is exceeded a safety device will automatically start-up. As a result the table might gradually move downwards regardless of the motor activation. Reduce the load in order to restore the proper operation of the table.*

Absolutely avoid leaning on the very edge for support as it may tilt over.

If pressure on a reduced area exceeds the allowed weight, the table upholstery could split up. Keep the patients from getting on by their knees. The maximum weight limit of every table section is 50 kg.

To **get on** the couch the patient must seat at the middle section. Then he can lean back.

When **getting down** the couch different procedures apply depending on the position of the patient. Therefore, if he is lying down (prone position) he has to slide gently towards the middle of the couch, lean on his side and lean forward by standing on his arms. He will then be able to put his legs out by the table's side and easily stand up. If the patient is lying upwards, raise the backrest and lower the leg section so that he gets back to a seated position enabling him to go down the couch.

In case the Chair is equipped with **armrests**: flip them up to get up / down the Chair easily.

**\* Caution! The patient must not lean on the armrest as it could split. This table part is not aimed at supporting such a pressure and it could split.**

**Notice:** The patient must always lean on the central section. Absolutely avoid leaning against the head or feet end for support because the couch may tilt over. If pressure on a reduced area exceeds the allowed weight, the table upholstery could split up. Keep the patients from getting on by their knees. The maximum weight limit of every table section is 50 kg.

### c) HEIGHT adjustment

The central height of the table is operated by an **electric motor**. To operate the supply cable must be plugged into a 220-230 V socket.

**When raising or lowering the stretcher or when operating its modules, MAKE SURE THAT NEITHER THE PATIENT NOR THE THERAPIST ARE TRAPPED** between the moving parts of the lifting mechanism by hands, feet or any other part of the body or clothing. Be careful, since there is a risk of entrapment between mobile and fixed parts of the stretcher, or, and due to unexpected movements when activating its modules. Only operate the stretcher when there is no possibility that any part of the body could be trapped and injured while moving the stretcher up/down or activating the tilt of the component modules.

**Pay special attention in the case of children and/or unauthorized personnel. Do not leave children alone or unattended in the area around the stretcher. Maintenance or cleaning tasks must be carried out only by authorized personnel and with the stretcher drive systems properly blocked.**

For adjusting the height use the **foot pedal**. Press the foot control switch pedal: left downwards / right upwards.

**Attention:** When adjusting the height, the stretcher makes a horizontal displacement of about 20 cm from its initial position. This requires a space that facilitates maneuverability during the process of adjusting the height of the stretcher. Take care during the process to avoid entrapment between the stretcher and the wall or adjacent objects.

### d) SECTIONS adjustment

The **head-/backrest** section is operated by a gas spring (3)

The **central section** is operated by a gas spring (3)

The **feet section** is operated by knob/lever (1)

(1) Knob/Lever: With one hand, loosen the knob or lever to release the bar. With the other hand, position the section at the required tilting. Release the knob or lever and the section will lock itself.

(3) Gas spring: Press the mechanism situated at the very edge of the section towards the upholstery and the table section will go up slowly. Release mechanism to lock position. To lower the section, press the mechanism with one hand and with the other hand, push the section downwards to the required position.

**NEVER adjust any of the table sections WHILE THE PATIENT IS LEANING ON. Safely position your FINGERS AND HANDS AWAY from the upholstery edge to avoid any accidental jamming.**

### e) ACCESORIES Adjustment

To lift the **ARMREST** simply pull it upwards from the edge.

**\*Notice:** The patient **must not lift by leaning on the armrest** as it could split following an excessive strain.

## 3) MAINTENANCE

For safety reasons, **DISCONNECT THE TABLE from the electrical supply** before the table cleaning, repair or servicing.

### **a) Metal FRAME**

For a proper operation of the chair, you should **do the dusting** regularly with a cloth.

Periodically **check** visually for any signs of damage and make sure that all bolts, nuts and other fasteners are tight.

*\* Please note that the repeated exposure of the table to high levels of humidity is not recommended.*

### **b) Electric MOTOR**

Keep this piece free from any dust or grease.

Absolutely avoid spilling liquids on the motor because they could cause severe damage to the electric components despite the motor protection.

**THE HANDLING OF THE ELECTRIC MOTOR IS TOTALLY FORBIDDEN. Any manipulation will invalidate the guarantee.**

A non-stop blinking operation of the foot rail or any other height adjustment mechanism could activate the thermal protection of the motor due to an overheating.

For further information, **go to annex "FREQUENTLY ASKED QUESTIONS ABOUT MOTORS"**.

Examine the table periodically for any signs of damage or malfunction and contact immediately the reseller or Ecopostural for service/ spare parts.

### **c) Section adjustment MECHANISMS**

Keep the mechanisms free from any dust or grease.

**THE HANDLING OF THE GAS SPRING AND/OR THE ELECTRIC MOTOR IS TOTALLY FORBIDDEN. Any manipulation will invalidate the guarantee.**

If any malfunction or failure is suspected, contact immediately the reseller or Ecopostural.

### **d) UPHOLSTERY**

A wet cloth should be enough to do the dusting. Anyway, if the leather is not properly clean you can use a smooth sponge soaked with neutral soap water. Dry immediately afterwards. To prevent moisture from entering the seams, do not wash with abundant water.

It is highly recommended to use a towel cover like the A4469, A4459 or A4460 to protect the upholstery from sweat, massage creams or oils.

Use soapy water to clean the stains. **Avoid using alcohol, any solvent or stain remover** as the vinyl could be seriously damaged.

The use of infrared lamps might cause a cracking of the artificial leather. You should then put a cotton sheet or towel on the couch.

**\* For further details please CHECK THE ENCLOSED UPHOLSTERY INSTRUCTIONS.**

## **ECOPOSTURAL GUARANTEE**

All our equipment is covered by a **3 years** guarantee, from the original invoice date, against any manufacturing flaw or mechanical defect. It will be thus necessary to keep the invoice or any other purchasing document as proof of guarantee.

Wear and tear of the artificial leather will not be covered by the guarantee.

The following conditions will invalidate your guarantee: any damage or deterioration caused by misuse (e.g. inappropriate leather cleaning), by an undue unwrapping with a cutter (e.g. upholstery rips and tears) or by accident.

In those cases where the guarantee applies, Ecopostural will repair or replace the faulty part free of charge and return it at our expense within the EU territory.

For full details of the guarantee, please contact your supplier.

**PRODUCT**  **MANUFACTURED BY ECOPOSTURAL, S.L.U. ESB12433991**  
Avda. Enrique Gimeno, 27 - 12006 Castellón de la Plana, **SPAIN**  
[www.ecopostural.com](http://www.ecopostural.com) / [info@ecopostural.com](mailto:info@ecopostural.com) / +34 964 20 01 53

The disposal of this product is subject to a specific EC Directive and it must be followed accordingly, as shown by the crossed skip symbol. Therefore, at the end of its useful life, you will have to apply to a specialized waste disposal centre.

Separate Collection of used products and packaging allows materials to be recycled and used again. Re-use of recycled materials helps prevent environmental pollution and reduces the demand for raw materials.

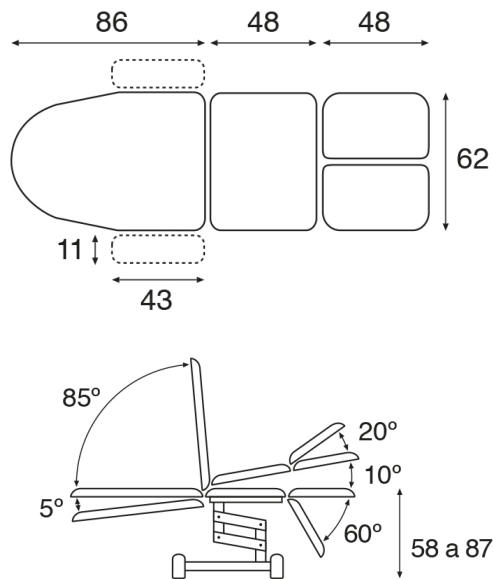
## FREQUENTLY ASKED QUESTIONS ABOUT MOTORS

### You should notice that:

The motor is equipped with a THERMAL PROTECTION device. A non-stop operation of the motor may cause an overheating thus triggering an automatic disconnection. The motor won't break down but it will need a while to cool down before restoring its activity. Simply stop operating the motor when reaching the maximum extend and the thermal protection won't be activated.

The motor is pneumatically ACTIVATED by means of an air flow going through an air pipe. Therefore, any alteration in the air pipe or in the environmental conditions may affect the proper operation of the motor.

| Problem:                                                                                                                                                                                                                                                                                         | Most common causes:                                                                                                                       | Solution:                                                                                                                                                                    |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p>The actuator (motor) is not moving.</p>                                                                                                                                                                     | The actuator is not power supplied.                                                                                                       | Check that the power point, the power cable and the different connections are properly plugged in                                                                            |
|                                                                                                                                                                                                                                                                                                  | The air pipe has been accidentally disconnected.                                                                                          | Connect the air pipe correctly.                                                                                                                                              |
|                                                                                                                                                                                                                                                                                                  | The air pipe has any holes or cuts, thus the air is leaking.                                                                              | Replace the air pipe.                                                                                                                                                        |
|                                                                                                                                                                                                                                                                                                  | The motor is hot. The thermal protection has been activated due to a non-stop operation of the motor or to a malfunction in the controls. | Unplug the table from mains.<br>Wait about 15 min. until the motor cools off.<br>If the problem persists, go to the section Controls not working properly'.                  |
|                                                                                                                                                                                                                                                                                                  | The load is excessive.                                                                                                                    | Reduce the load.<br>If the load is within the weight allowance and the actuator does not move anymore, <b>contact your supplier</b> (check first the whole troubleshooting). |
| <p>The controls do not work properly<br/>or<br/>The motor is activated without previously operating the controls.</p> <br> | The controls, the pipe and the actuator are exposed to sudden changes in temperature.                                                     | Level the pressure in the control system by disconnecting the air pipe, wait a minute and re-connect it.                                                                     |
|                                                                                                                                                                                                                                                                                                  | The controls, the pipe and the actuator are exposed to sudden changes in atmospheric pressure.                                            | Level the pressure in the control system by disconnecting the air pipe, wait a minute and re-connect it.                                                                     |
|                                                                                                                                                                                                                                                                                                  | The air pipe is trampled or has any knots and bends.                                                                                      | Unwind the air pipe to undo the effect of any bend, knot...<br>If the initial condition of the pipe is not restored it will have to be replaced.                             |
|                                                                                                                                                                                                                                                                                                  | The electronic switch is stuck thus continuously activating the actuator.                                                                 | Check that the foot rail system is in good condition. Contact your supplier for precise instructions.                                                                        |
|                                                                                                                                                                                                                                                                                                  | The control / switch is out of order.                                                                                                     | Replace the control / switch.                                                                                                                                                |
| The table moves downward without activating the motor.                                                                                                                                                                                                                                           | The load is excessive.                                                                                                                    | Reduce the load.<br>If the load is within the weight allowance and the tables keeps on going down, <b>contact your supplier</b> (check first the whole troubleshooting).     |

**TECHNICAL FEATURES C3567****General characteristics**

Electric white frame speciality couch.

Standard upholstery dimensions: 62x188 cm (M44)

Recommended maximum weight: 135 Kg

Height variation: 58-87 cm

Electric height adjustment motor controlled by foot pedal.

Height adjustment with side misalignment < 10cm

Steel frame with white epoxy powder coating heated at 250°C

Retractable castors with foot lever. Just for moving the table around, NOT intended for patient transportation.

Standard backrest type T19 - Rounded backrest. Without breather hole.

Neck bolster pillow (A4434) included as standard.

Set of armrests (non-removable)

Head-/Backrest adjustable by gas spring.

Central section adjustable by gas spring.

Feet section adjustable manually, by knob/lever.

Approx. weight: 65 Kg

**Upholstery Features**

Fire-resistant Class M2 artificial leather, EC compliant.

Sanitized® anti-microbial and antibacterial protection.

Available in 25 colours.

Foam thickness 5 cm.

Foam firmness II - Soft

**Electric Motor Features**

Stroke length: 150 mm

Asynchronous motor 1-230V. AC ±10%

Integrated thermal protection

Max. Power: 4000N - Average speed at load 0: 6mm/s

Voltage: 230V/50-60 Hz - Power consumption: 1,5A

Working cycle: 25s non stop / 400s off

Safety class: II - Standard protection: IPX4

EMI (electromagnetic interference) Directive compliant

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